


Quick Start Guide

seca mVSA 535

- Sicherheitshinweise • Safety precautions
- Consignes de sécurité • Avisos de seguridad


1.

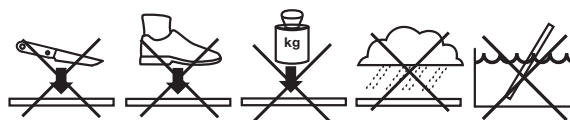


2.



- Lieferumfang (Zubehör variantenabhängig) • Scope of supply (accessories depend on version)
- Éléments livrés (accessoires selon le modèle) • Volumen de suministro (dependiendo de la variante del accesorio)

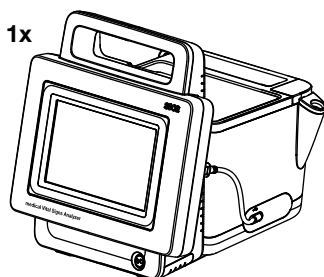


1x

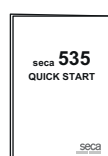


1x




1x

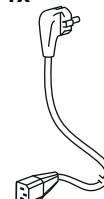


1x

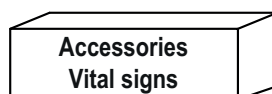


1x

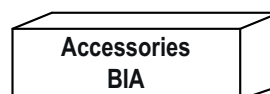


- Blutdruckmanschette anschließen (variantenabhängig) • Connecting the blood pressure cuff (depending on version)
- Raccorder le brassard de tensiomètre (selon le modèle)
- Conectar el manguito de presión arterial (dependiendo de la variante)

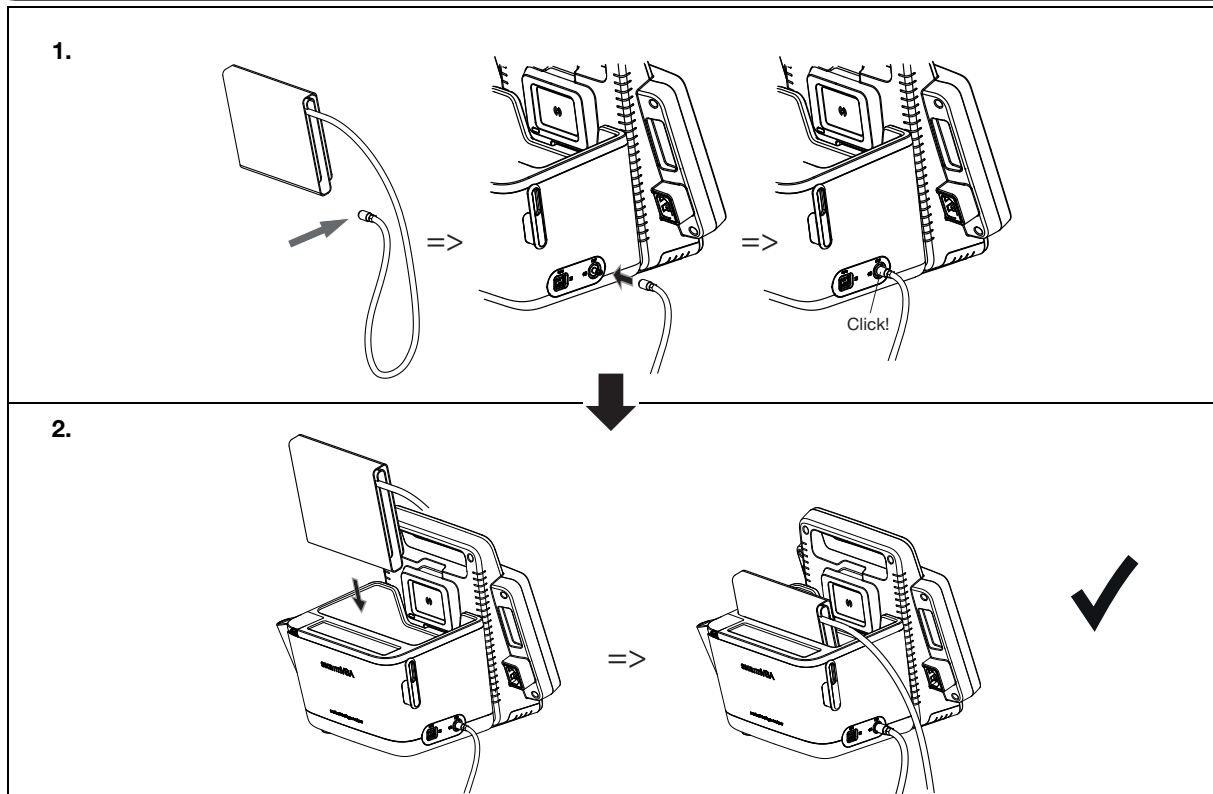


- Temperatursonde anschließen (variantenabhängig) • Connecting the temperature probe (depending on version)
- Raccorder la sonde de température (selon le modèle) • Conectar la sonda de temperatura (dependiendo de la variante)

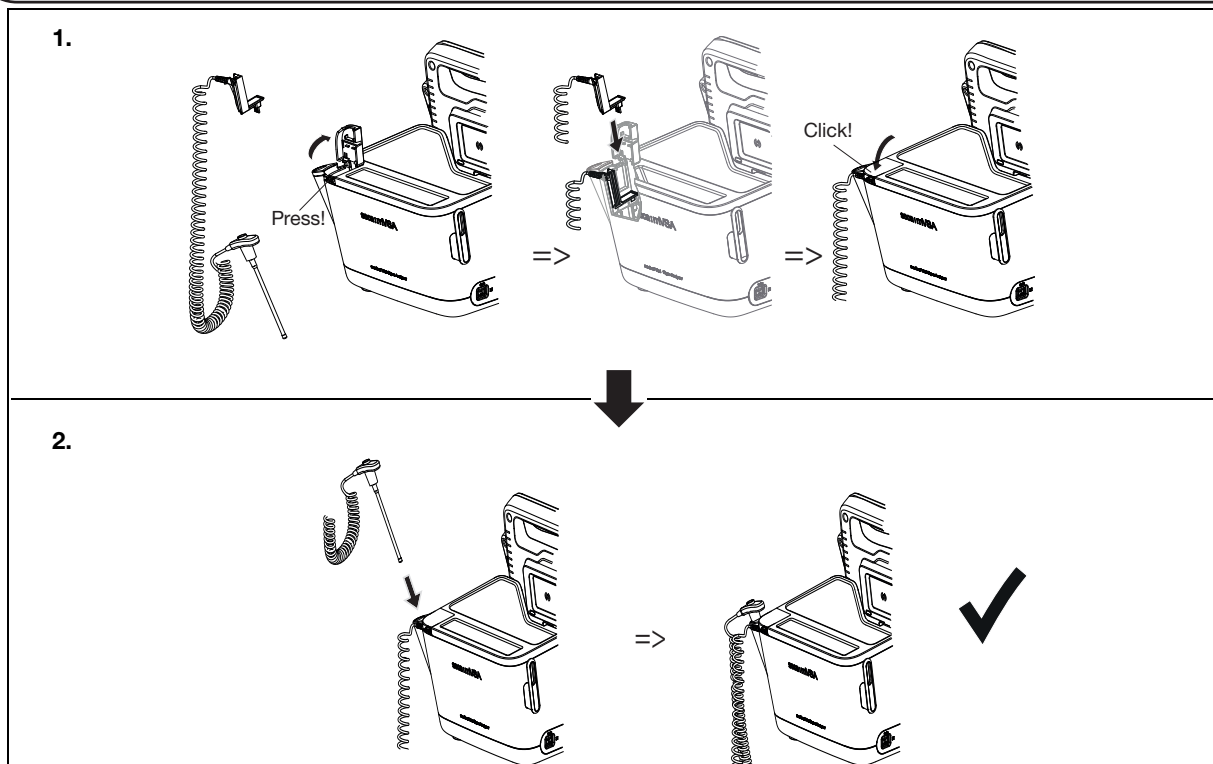


- Ohrthermometer anschließen (variantenabhängig) • Connecting the ear thermometer (depending on version)
- Raccorder thermomètre intra-auriculaire (selon le modèle) • Conectar el termómetro de oído (dependiendo de la variante)

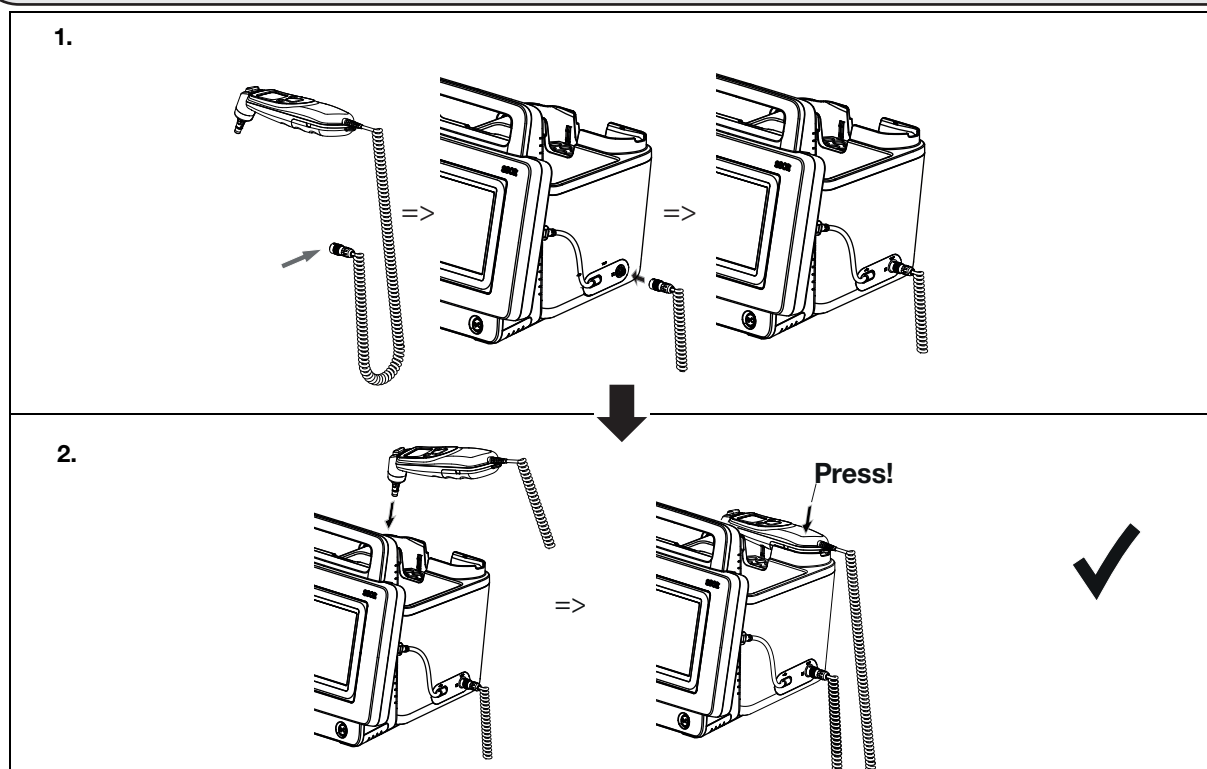


- SpO2-Sensor anschließen (variantenabhängig) • Connecting the SpO2 sensor (depending on version)
- Raccorder le capteur SpO2 (selon le modèle) • Conectar el sensor de SpO2 (dependiendo de la variante)

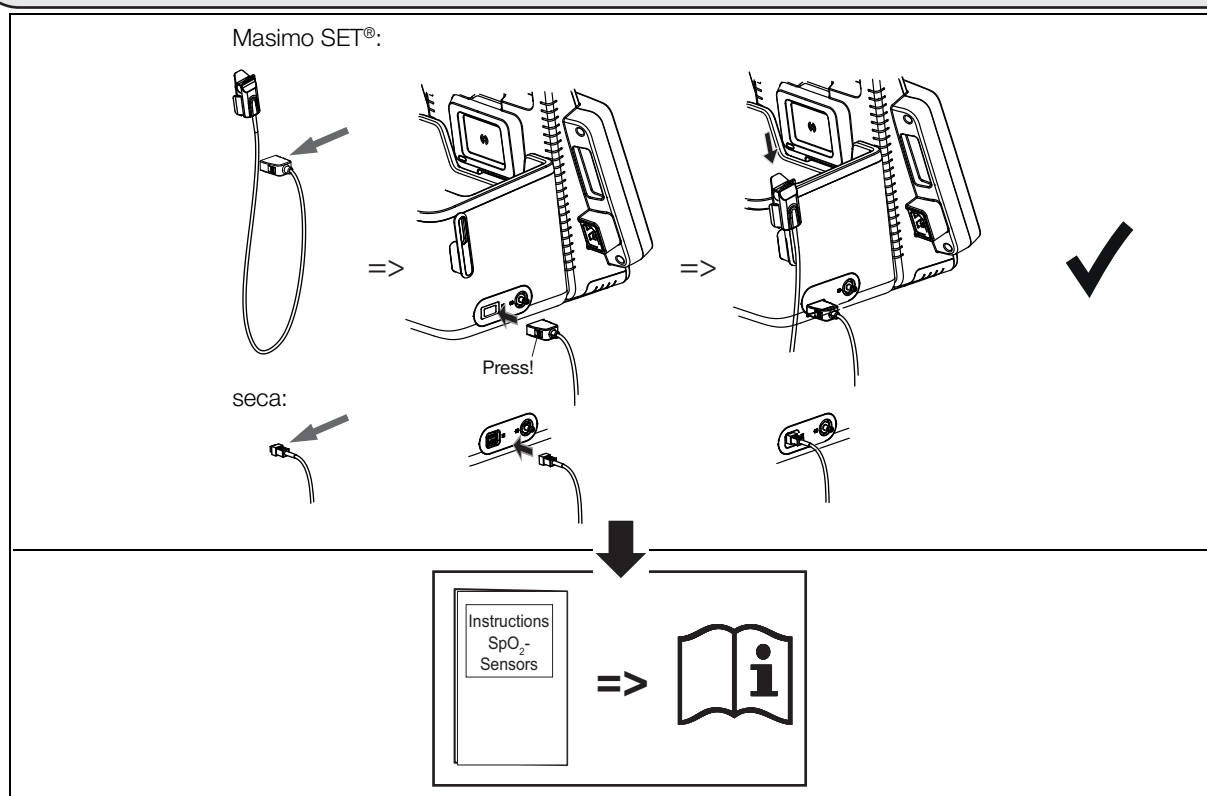


- Messmatte einhängen (variantenabhängig) • Placing measuring mat (depending on version)
- Accrocher la toise-tapis (selon le modèle) • Colgar cinta métrica (dependiendo de la variante)

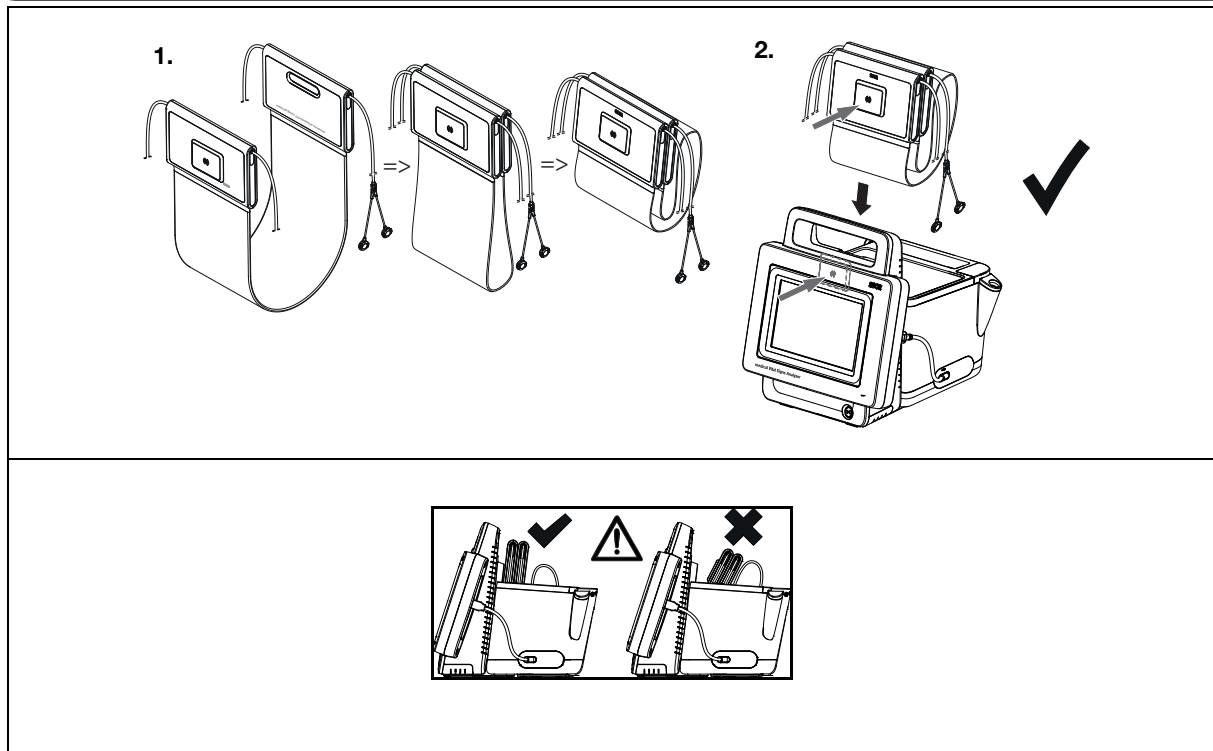





- Akkus laden • Charging Batteries
- Charger les piles • Cargar las baterías

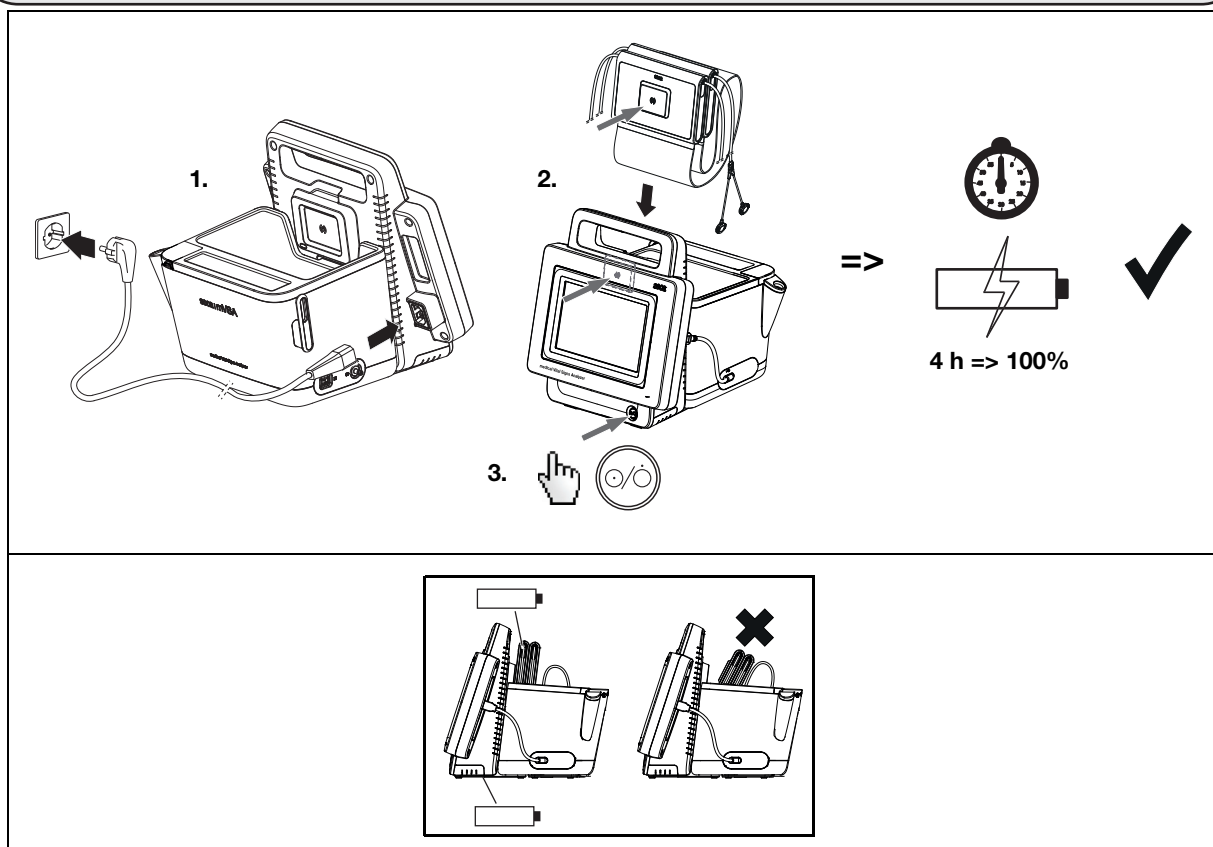


- Erster Systemstart: Datum/Uhrzeit einstellen • Initial system start: set Date/Time
- Démarrage initial du système : régler la date/l'heure • Primer inicio del sistema: ajustar fecha/hora

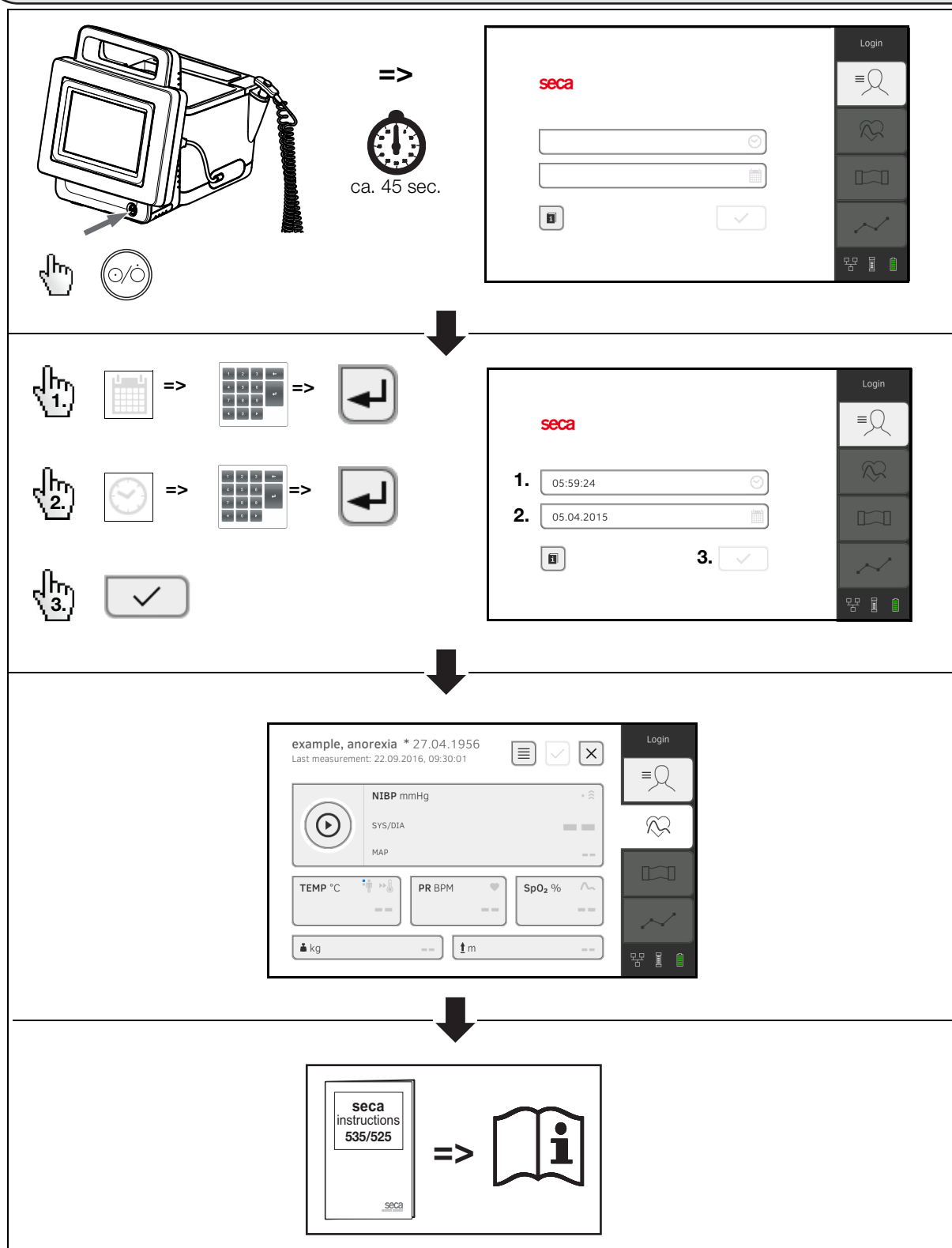


• Login • Login •
• Login • Iniciar sesión

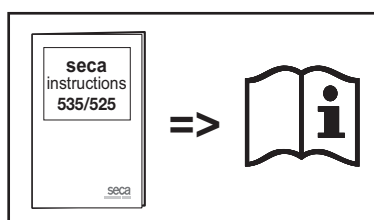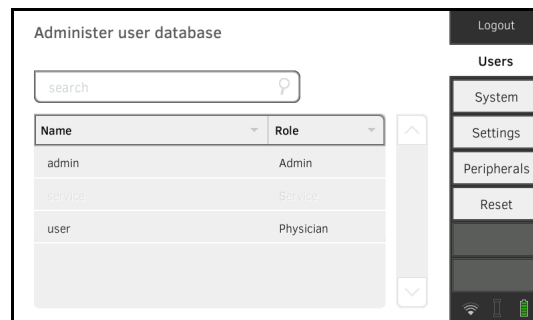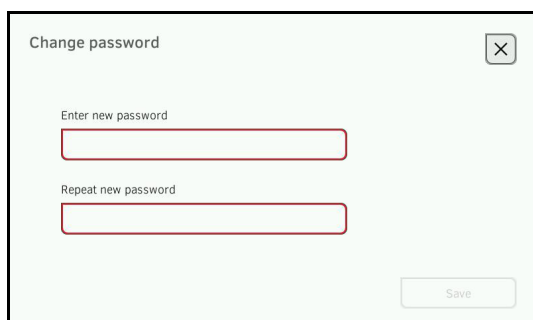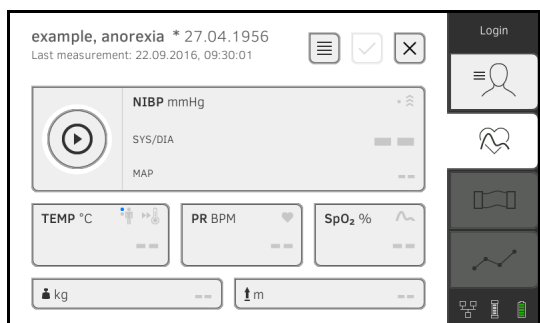

- Gebrauchsanweisung ansehen • View instructions for use
- Accéder au mode d'emploi • Consultar las instrucciones de uso

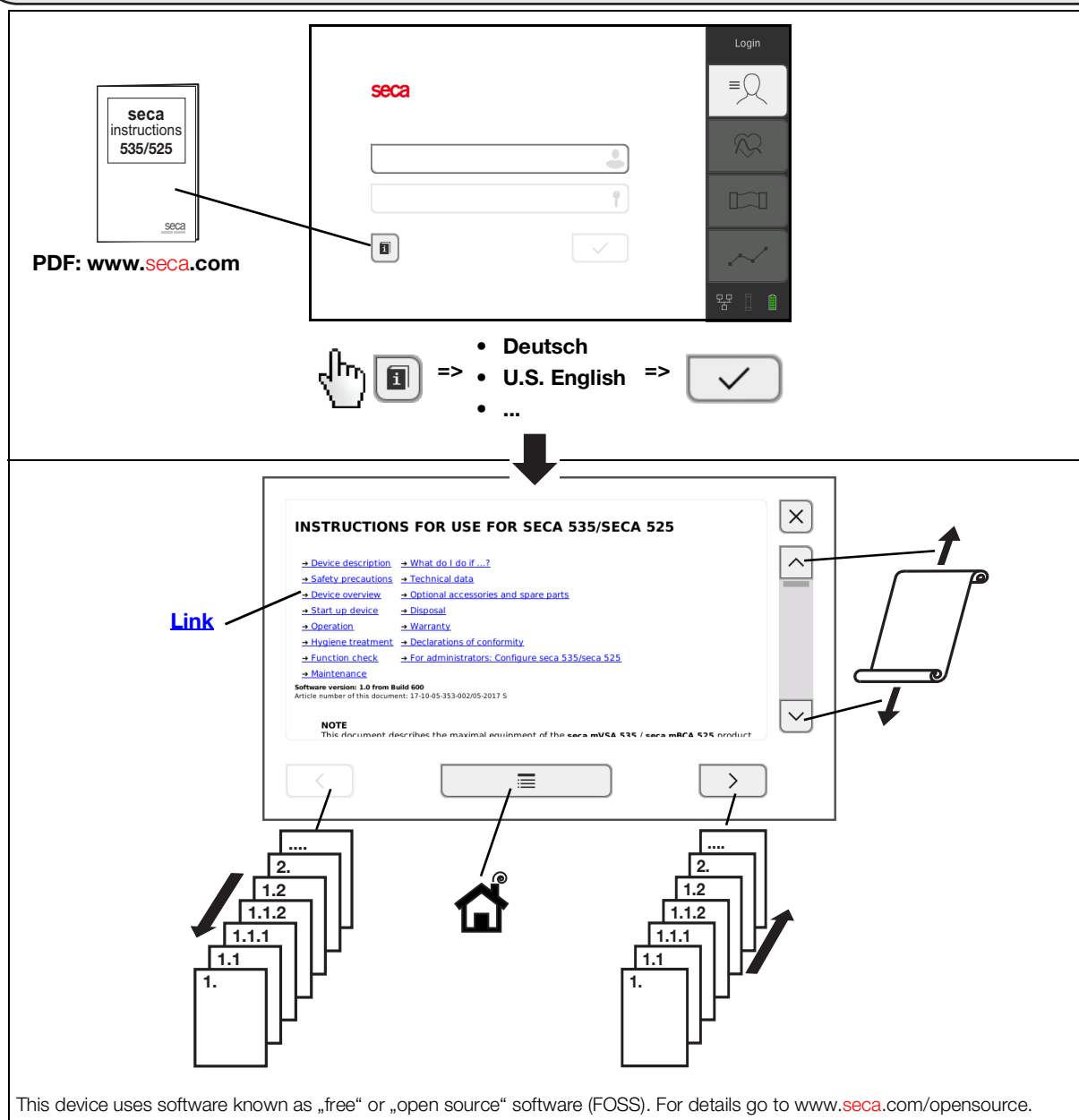

- seca analytics 115 installieren • Installing seca analytics 115
- Installer seca analytics 115 • Instalar seca analytics 115

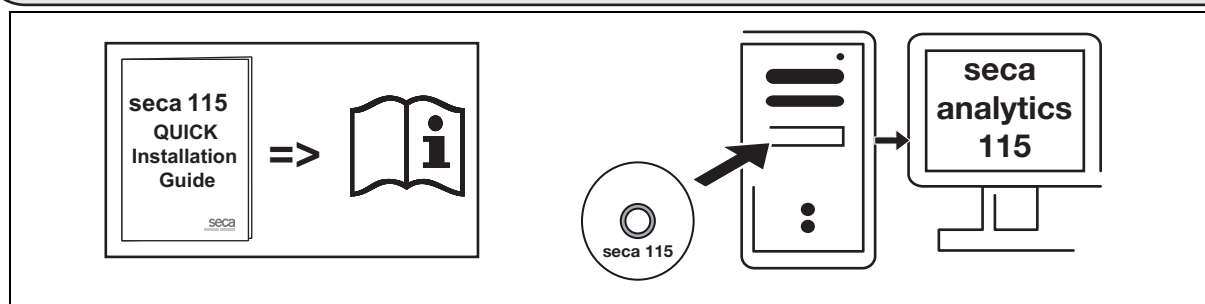

Medical Measuring Systems and Scales since 1840

seca gmbh & co. kg
Hammer Steindamm 3–25
22089 Hamburg · Germany
T +49 40 20 00 00 0
F +49 40 20 00 00 50
E info@seca.com

seca operates worldwide with headquarters
in Germany and branches in:

[seca france](#)
[seca united kingdom](#)
[seca north america](#)
[seca schweiz](#)
[seca zhong guo](#)
[seca nihon](#)
[seca mexico](#)
[seca austria](#)
[seca polska](#)
[seca middle east](#)
[seca suomi](#)
[seca américa latina](#)
[seca asia pacific](#)
[seca danmark](#)
[seca benelux](#)
[seca lietuva](#)

and with exclusive partners in
more than 110 countries.

All contact data at seca.com