POWER SUPPLY FOR THIS PRODUCT

In contradiction to what is stated in the operating instructions, this product is in fact supplied with rechargeable batteries and an external charger.

Please heed the following safety instructions:

ATTENTION!

Short runtime

The rechargeable batteries are supplied partially charged.

- Fully charge the batteries before using the device for the first time
- Take note of the operating instructions for the external charger also supplied.

ATTENTION!

Self-discharge

If the device is not used for a long time, the batteries will discharge themselves.

- Fully charge the batteries at regular intervals (recommended interval: 4 weeks).
- Take note of the operating instructions for the external charger also supplied.

NOTE:

The number of possible weighing operations specified in the operating instructions is based on operation with non-rechargeable batteries. When using rechargeable batteries, the maximum number of possible weighing operations is significantly lower. Recharge the batteries as soon as the  symbol appears on the display.

HANDLING (RECHARGEABLE) BATTERIES

WARNING!

Personal injury with improper handling

(Rechargeable) batteries contain harmful substances which may explode and so escape if not handled properly.

- Do not try to recharge disposable batteries.
- Do not expose (rechargeable) batteries to heat.
- Do not burn (rechargeable) batteries.
- If acid is leaking out, avoid contact with the skin, eyes and mucous membranes. Rinse affected areas with plenty of clean water and seek medical help at once.

ATTENTION!

Damage to device and malfunctions with improper handling

- Only ever use (rechargeable) batteries of the type "AA".
- When replacing (rechargeable) batteries, always replace a complete set at a time.
- Do not short-circuit (rechargeable) batteries.
- If you do not use the device for a long period of time, remove the batteries (incl. rechargeable batteries). This prevents acid from leaking into the device.

Spent (rechargeable) batteries should not be discarded with household waste, regardless of whether they contain harmful substances or not. As a consumer you are obliged by law to dispose of (rechargeable) batteries via the collection points set up by the municipal authorities or the retail sector. Only discard (rechargeable) batteries when fully discharged.

seca gmbh & co. kg
Hammer Steindamm 9-25
22089 Hamburg · Germany

Telephone · +49 (0)40 20 00 00 0

Fax · +49 (0)40 20 00 00 50

E-mail · info@seca.com

All contact details available at www.seca.com

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